FR

REF WMOEV10

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REF WMOEV10

Features

The Osto-EZ-Vent® is a pouch venting device which can be easily attached to the film of an ostomy pouch. It can be opened and closed and, with a hole made in the pouch with scissors using the Osto-EZ-Vent® exhaust hole as a guide, can be used to vent gas quickly from the pouch, thereby relieving the situation of 'ballooning'.

Intended Use

The Osto-EZ-Vent® is an accessory product designed to be used with ostomy appliances. Its purpose is to allow easy and quick venting of an ostomy bag to expel

Osto-EZ-Vent® Vent Ostomy pouch venting device

the flatus that builds up within the bag, often overnight. This situation is known as 'ballooning' and affects many ostomates.

Materials

The Osto-EZ-Vent® is made from low density polyethylene. On the rear face of the circular section is an area of adhesive coated foam, provided with a vent hole. Release film covers the adhesive and is easily removable at the time of use to allow the user to adhere the Osto-EZ-Vent® to the pouch film.

Application of the Osto-EZ-Vent®:

Step 1: Prepare pouch and Osto-EZ-Vent®

- The Osto-EZ-Vent® should be attached nearest the top on the non-stoma side of the pouch.
- For covered pouches carefully cut a small hole in the cover to attach the Osto-EZ-Vent® directly to the film of the pouch [Photo 1].
- It is advised to clean the film surface area where the Osto-EZ-Vent® will be attached with a disinfecting wipe or alcohol solution on a cotton pad and let the area dry before attaching the Osto-EZ-Vent® [Photo 2].

Step 2: Place Osto-EZ-Vent® on pouch

- Peel the green/white release film from the Osto-EZ-Vent®. Do not touch the adhesive [Photo 3].
- Once the cleaned area has dried, place the Osto-EZ-Vent® horizontally on the pouch [Photo 4].
- With thumb and forefinger, press firmly all around the Osto-EZ-Vent® for maximum adhesion.

Step 3: Make Osto-EZ-Vent® hole in pouch

- Once the Osto-EZ-Vent® is securely bonded to the pouch, pull the top of the pouch where the Osto-EZ-Vent® is attached down through the stoma opening, insert narrow blade of scissors through the Osto-EZ-Vent® opening and make a hole the same size as the Osto-EZ-Vent® opening to allow air to escape easily [Photo 5].
- Once the hole is made, snap the Osto-EZ-Vent® closed. The pouch is ready for use [Photo 6 & 7].
- By pinching the glides, the glides push against the base to help lift up and open the Osto-EZ-Vent® [Photo 8].

Disposal

When removing your pouch leave the Osto-EZ-Vent® in place and dispose of it in the household waste. Do not flush your appliance down the toilet as this can block your sewage system.

Warnings and Precautions

- Read the instructions for use prior to use.
- Each Osto-EZ-Vent® is for single use only. Do not re-use. Re-use can lead to cross contamination.
- Do not remove the release film until ready to adhere the Osto-EZ-Vent® to the pouch.
- Attach the Osto-EZ-Vent® to the pouch 24 hours before using. Full adhesion is achieved after 24 hours. The Osto-EZ-Vent® can be pre-attached when you receive your supplies.
- Ensure that the Osto-EZ-Vent® is well adhered to the pouch before wearing and that the lid is closed.
- Take care only to cut the film as directed in Step 3 above when making the vent hole.
- Ensure to always close the Osto-EZ-Vent® after venting gas from the pouch.
- Care should be taken when venting to ensure no seepage of stools through the Osto-EZ-Vent®.

Storage Conditions

It is best to store your products in a cool dry place with a guidance temperature from 10°C to 25°C as is shown on the product label.

Contraindications

- Do not use if you have a known allergy to any of the materials used in the Osto-EZ-Vent®.
- Poor manual dexterity or with visual impairment - do not use the vent without assistance.

Product	Quantity	Article Number
Osto-EZ-Vent®	10 pieces	WMOEV10